

# Impella CP™ with SmartAssist™ Heart Pump

## Confident Positioning. Simplified Set-up.

### 8<sup>th</sup> Generation

#### Enhanced Pigtail

Tapered (kink resistant) base made of a single material with gradual change in stiffness.



### 9<sup>th</sup> Generation



#### Simplified Set-Up

Reduction in set-up steps due to repositioning unit moved back on the catheter, no need to pull back during setup



#### Modular Purge Cassette

Clean design, for simple and quick setup simplifies line management by reducing the amount of purge tubing and components



**Do Not remove the extension line tubing!**  
The Impella CP™ with SmartAssist™ pump requires the extension line tubing

#### Purge Fluid

Simplified anticoagulation management possible, enabled through sodium hydrogen carbonate (Bicarb/NaBic) in purge solution

Glucose G5% with 25 mmol/L (mEq/L) of Sodium hydrogen carbonate (Bicarb or NaBic)\*

\*G5% — preferred; G5%–G20% acceptable;  
G5% with heparin 25 U/mL or 50 U/ml is also acceptable.



#### Impella® Device Indication & Safety Information

To learn more about the Impella platform of heart pumps, including important risk and safety information associated with the use of the devices, please visit <https://www.heartrecovery.eu/safety-information>

- or -



Scan code to learn more



**Abiomed, Inc.**  
22 Cherry Hill Drive  
Danvers MA 01923 USA  
Phone: +1 978-646-1400  
Fax: +1 978-777-8411  
marketing@abiomed.com



**Abiomed Europe GmbH**  
Neuenhofer Weg 3  
52074 Aachen, Germany  
Phone: +49 (0) 241 8860-0  
Fax: +49 (0) 241 8860-111  
europe@abiomed.com

[www.heartrecovery.com](https://www.heartrecovery.com)

© Abiomed Europe GmbH 2025